

**Variolink®** Esthetic  
Adhesive Cement System



**“Esthetic Cementation  
made Simple!”**

*The adhesive cement for  
exceptional esthetics and  
user-friendly application...*



**PATENTED**  
**Ivocerin**  
LIGHT INITIATION

**ivoclar  
vivadent:**  
passion vision innovation

# CLEAN



## Clean Restoration

- Rinse restoration with water and air dry.
- Cover entire bonding surface with Ivoclean®.
- Allow to react for 20 seconds. Rinse and air dry.



# PRIME



## Prime Restoration

- Apply Monobond Plus® to the bonding surface of the restoration.
- Allow to react for 60 seconds. Air dry.



# BOND



## Bond Restoration

- Bond with Variolink® Esthetic



## Simplified Bonding



**Step 1 - Prep Dentin/Enamel**

- Total Etch optional
- Apply and scrub **Adhese® Universal** into preparation for 20 secs.
- Thin with air
- Light cure for 10 seconds



**Step 2 - Cement**

- Dispense **Variolink Esthetic** into restoration and seat.



**Step 3 - Clean-Up**

- After seating, light-cure each quarter surface for 2 seconds. The cement will achieve a gel-like consistency for easy clean-up.
- Light cure buccal and lingual margins for 20 seconds each.



**Step 4 - Post-op**

- Use polishing points to finish margins.

## DELIVERY FORMS

### Variolink Esthetic LC System Kit (Light-cure)

- 3 Syringes Variolink Esthetic LC, 2 g (light, neutral, warm)
- 3 Syringes Variolink Esthetic Try-In Paste, 1.7 g (light, neutral, warm)
- 1 Syringe Total Etch, 2 g
- 1 Adhese Universal VivaPen, 2 ml, or 1 bottle, 5 g
- 1 Bottle Monobond Plus, 5 g
- 1 Syringe Liquid Strip, 2.5 g
- Various accessories

- 666433 with Adhese Universal VivaPen, 2 ml
- 666065 with Adhese Universal bottle, 5 g

### Variolink Esthetic DC System Kit (Dual-cure)

- 3 Automix syringes Variolink Esthetic DC, 5 g (light, neutral, warm)
- 3 Syringes Variolink Esthetic Try-In Paste, 1.7 g (light, neutral, warm)
- 1 Syringe Total Etch, 2 g
- 1 Adhese Universal VivaPen, 2 ml, or 1 bottle, 5 g
- 1 Bottle Monobond Plus, 5 g
- 1 Syringe Liquid Strip, 2.5 g
- Various accessories

- 666434 with Adhese Universal VivaPen, 2 ml
- 666125 with Adhese Universal bottle, 5 g

### Variolink Esthetic LC Refill (Light-cure)

- 1 Syringe, 2 g (in the selected shade)
- 5 Application tips

- 666126 Light+
- 666127 Light
- 666128 Neutral
- 666129 Warm
- 666130 Warm+



### Variolink Esthetic DC Refill (Dual-cure)

- 1 Automix syringe, 5 g (in the selected shade)
- 10 Mixing tips

- 666117 Light+
- 666118 Light
- 666119 Neutral
- 666120 Warm
- 666121 Warm+

### Variolink Esthetic DC Refill (Dual-cure)

- 1 Automix syringe, 9 g (in the selected shade)
- 15 Mixing tips

- 666122 Light
- 666123 Neutral
- 666124 Warm

### Variolink Esthetic Try-In Paste Refill

- 1 Syringe, 1.7 g (in the selected shade)
- 5 Application tips

- 666067 Light+
- 666113 Light
- 666114 Neutral
- 666115 Warm
- 666116 Warm+

### Accessories

- 645951 Mixing tips, short, refill, 15 units
- 559224 Variolink application tips, refill, 20 units

## FRIENDSHIP DENTAL LABORATORIES, INC.

8510 Philadelphia Road  
Rosedale, MD 21237

Phone: 410-780-7700

Email: [info@friendshipdentallab.com](mailto:info@friendshipdentallab.com)

[WWW.FRIENDSHIPLABS.COM](http://WWW.FRIENDSHIPLABS.COM)



**ivoclar  
vivadent:**  
passion vision innovation